

THANK YOU FOR PURCHASING**THE 1301-88 PELVIS FOR FRACTURE DEMONSTRATION.**

This product will last many years with some basic cleaning and proper storage. To ensure you get the full life out of the model, only use the erasable pen that is provided. Use of a Sharpie or similar product may discolor the model or may not be removable from the bone surface.

Over time you may notice some stubborn dark spots or smudging that may be difficult to remove with a dry cloth.



This can be easily remedied with warm soap and water and the use of a toothbrush or similar scrubbing device to remove the dark spots. This procedure will not harm the model.

**STORAGE**

When the model is not in use it is recommended that it be stored out of direct sunlight and do not store it in temperatures above 120° F.