




## THANK YOU FOR PURCHASING SAWBONES TRUNK FOR PARACENTESIS AND THORACENTESIS PROCEDURES

Soft tissue trunk with replaceable anterior and posterior inserts with fluid for extraction.

### COMPONENTS

- Soft tissue trunk
- Replaceable anterior and posterior inserts with fluid for extraction


**NOTE:** Replacement parts available at [www.sawbones.com](http://www.sawbones.com)

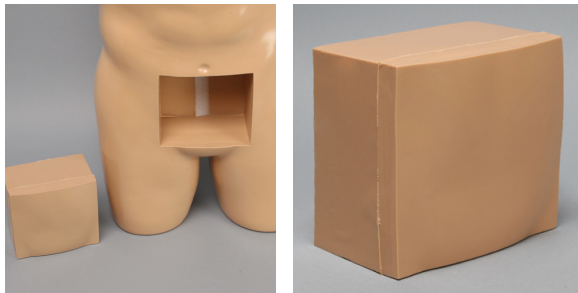

**1513-36-2**  
Paracentesis Replacement  
Insert

### PARACENTESIS PROCEDURE

1. The model has a self contained, self sealing water reservoir (14 x 15.5 cm) insert with a 500 cc fluid capacity.
2. Model includes skin, fat layer, rectus abdominis muscles, urachus and peritoneal cavity.
3. Water reservoir comes filled with fluid.
4. After removing fluid from reservoir, reverse the procedure to refill the insert from the syringe. \*

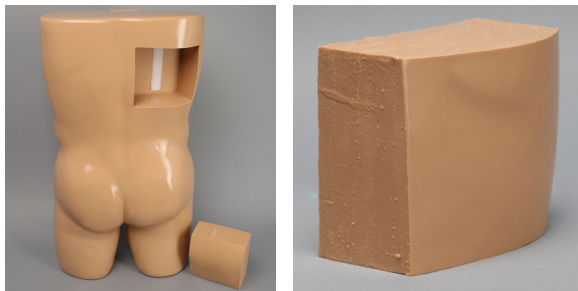

**1513-36-1**  
Thoracentesis  
Replacement Insert

### THORACENTESIS PROCEDURE

1. The model has a self contained, self sealing water reservoir (14 x 15.5 cm) insert with a 400 cc fluid capacity.
2. Model includes skin, fat, muscle, ribs, parietal pleura and lung cavity (water reservoir).
3. Water reservoir comes filled with fluid.
4. After removing fluid from reservoir, reverse the procedure to refill the insert from the syringe. \*

\* The insert should be able withstand 40 to 50 sticks before needing to be replaced. Often times more, depending on the gauge of the needle.