




## REQUIRED ITEMS

In addition to the items included in the Portable Arthroscopy Camera (PAC) System, you will also need:

- A computer or tablet with two Powered USB Type A ports.

**NOTE:** Not compatible with iPads

## VIDEO SET-UP INSTRUCTIONS

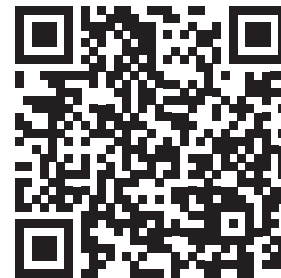

## SET-UP INSTRUCTIONS




1. Remove the lens cap of the camera c-mount coupler by rotating the coupler's quick connect/release.



2. Attach the eyepiece of the arthroscope to the camera c-mount coupler by rotating the coupler's quick connect/release.





3. Attach the light to the scope by pressing its quick connect/release at the top of the light.

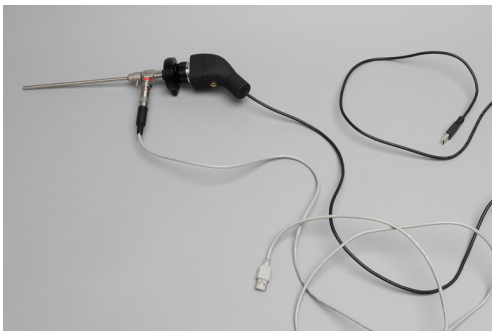

4. Plug in both USB connectors into the USB ports on your computer or tablet (Two powered USB Type A ports are required). **NOTE:** Not compatible with iPads

5. Turn on your computer or tablet and open your native camera app. Switch to the correct camera if multiple are connected (e.g. webcams). To do so in Windows:

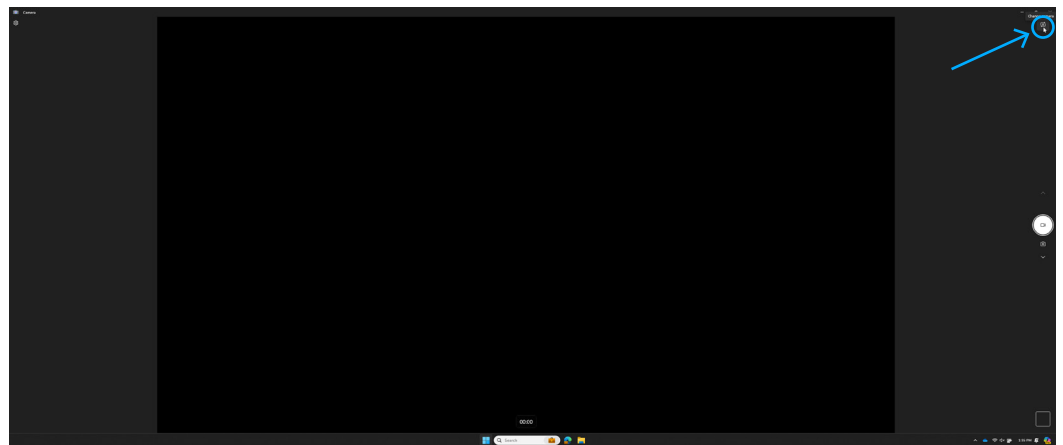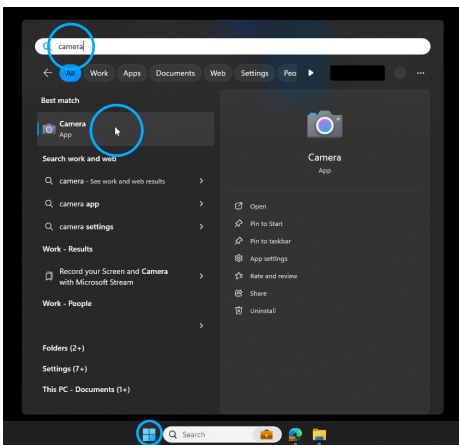

Press the Windows key on your keyboard, type "camera", and select the camera app.

If you see yourself, the computer is using the built in camera. Click the icon on the top right to switch cameras until you see what is being captured from the arthroscope.

6. Adjust the manual focus if needed.